

MLAB 2360 Clinical II  
Activity 19: HTR

PRE-TRANSFUSION SAMPLE

Name <b>Johnson, Joseph</b>											
ID #	51275133		Anti A	Anti B	Anti D	D Ctrl	Wk D	Wk D ctrl	A <sub>1</sub> cells	B cells	Interp
			4+	0	4+				0	4+	A pos
Retype											
			4+	0	4+						

BB#	Rack#	Collected	Completed	Tech	IS			37°			AHG			Check Cells			Interp.
					S1	S2	S3	S1	S2	S3	S1	S2	S3	S1	S2	S3	
WKR 4123	2	Date: 4/20 Time: 0530	Date: 4/20 Time: 0600	TK	0	0	0	0	0	0	0	0	0	2+	2+	2+	Neg

SPECIAL TESTING											
DAT						FBS			Antibody Identification		
Poly	IgG	Interp	C <sup>3</sup>	Interp	Interp	Pos	Neg	Pos	Neg	Pos	Neg

Blood Bank #	Donor #	Anti-A	Anti-B	Anti-D	Interp ABO/D	IS	37	AHG	Interp	Tech	Date	Time
1	WKR 4123	4+	0	4+		0	0	0	2+	TK	4/20	0600
2	WKR 4123	4+	0	4+		0	0	0	2+	TK	4/20	0600
3												
4												

Comments
----------

MLAB 2360 Clinical II  
Activity 19: HTR

POST-TRANSFUSION SAMPLE

Name <b>Johnson, Joseph</b>											
ID #	51275133		Anti A	Anti B	Anti D	D Ctrl	Wk D	Wk D ctrl	A <sub>1</sub> cells	B cells	Interp
			0	0	4+				4+	4+	O pos
Retype											
			0	0	4+						

BB#	Ract#	Collected	Completed	Tech	IS	37°			AHG			Check Cells			Interp.
		Date:	Date:		S1	S2	S3	S1	S2	S3	S1	S2	S3		
RVS	2	4/20	4/20	JE											
1143		Time: 1000	Time: 1045		0	0	0	0	0	0	2+	2+	2+	Neg	

SPECIAL TESTING													
DAT						FBS			Antibody Identification				
Poly	✓	Interp	IgG	✓	Interp	C <sup>3</sup>	✓	Interp	Pos	Neg	Patient		
2+		Pos	2+		Pos	1+	2+	Pos					

Blood Bank #	Donor #	Anti-A	Anti-B	Anti-D	Interp ABO/D	IS	37	AHG	✓	Interp	Tech	Date	Time
1	WKR 4123	4+	0	4+		4+	4+	4+		Inomp	JE	4/20	1045
2	WKR 4123	4+	0	4+		4+	4+	4+		Iomp	JE	4/20	1045
3													
4													

Comments  
Visible hemolysis in post-transfusion sample, no hemolysis in pre-transfusion sample.  
Clerical check between pre- and post-transfusion samples - no discrepancy observed.

MLAB 2360 Clinical II  
Activity 19: HTR

Name <b>Smith Steven</b>									
ID #	51275133								
Retype									
Anti A	Anti B	Anti D	D Ctrl	Wk D	Wk D ctrl	A <sub>1</sub> cells	B cells	Interp	
0	0	4+			✓	4+	4+	O pos	

BB#	Rack#	Collected	Completed	Tech	IS			37°			AHG			Check Cells			Interp.
					S1	S2	S3	S1	S2	S3	S1	S2	S3	S1	S2	S3	
RVS	2	Date: 4/20 Time: 1000	Date: 4/20 Time: 1045	JE	S1	S2	S3	S1	S2	S3	S1	S2	S3	S1	S2	S3	Pos
1143					3+	0	0	2+	0	0	1+	0	0	2+	2+	2+	

SPECIAL TESTING											
DAT						FBS			Antibody Identification		
Poly	✓	Interp	IgG	Interp	C <sup>3</sup>	Pos	Neg	Pos	Interp	Patient	
2+		Pos	2+	Pos	1+	2+					

Blood Bank #	Donor #	Anti-A	Anti-B	Anti-D	Interp ABO/D	IS	37	AHG	Interp	Tech	Date	Time
1	WKR 4123	0	0	4+		0	0	0	2+	TK	4/20	1045
2	WKR 4125	0	0	4+		3+	2+	1+	2+	TK	4/20	1045
3												
4												

Comments



MLAB 2360 Clinical II  
Activity 19: HTR

Name <b>Phillips, Martha</b>										Interp									
ID #	4758445		Anti A	0	Anti B	0	Anti D	4+	D Ctrl	Wk D	✓	Wk D ctrl	✓	A <sub>1</sub> cells	4+	B cells	4+	Interp	O pos
Retype																			

BB#	Rack#	Collected	Completed	Tech	IS			37°			AHG			Check Cells			Interp.	
					S1	S2	S3	S1	S2	S3	S1	S2	S3	S1	S2	S3		
XTR 1223	2	Date: 4/20 Time: 1000	Date: 4/20 Time: 1045	JE	0	0	0	0	0	0	0	0	0	0	0	2+	2+	Pos

SPECIAL TESTING											
DAT						FBS			Antibody Identification		
Poly	IgG	Interp	C <sup>3</sup>	Interp	Interp	Pos	Neg	Patient	Antibody Identification		
0	2+	Pos	0	2+	Neg						

Blood Bank #	Donor #	Anti-A	Anti-B	Anti-D	Interp ABO/D	IS	37	AHG	✓	Interp	Tech	Date	Time
1. WKR 4123	117488	0	0	4+		0	0	3+	2+	incomp	TK	4/20	1045
2. WKR 4123	1154779	0	0	4+		0	0	0	2+	comp	TK	4/20	1045
3													
4													

Comments

