



## Step-by-Step Guidelines

# Portex® Venipuncture Needle-Pro® Device

**IMPORTANT:** Please refer to the instructions for use supplied with the product for detailed instructions, warnings, and cautions.



- 1 Secure needle into Venipuncture Needle-Pro® device. Rotate until firmly seated.



- 2 Insert tube into holder. Advance no further than the guideline on the holder.



- 3 Remove needle sheath and rotate orange needle protection device away from needle bevel or to desired position prior to venipuncture.



- 4 Perform venipuncture as per normal procedure.



- 5 Upon completion of specimen collection, engage the Venipuncture Needle-Pro® device using a one-handed technique by gently pressing the sheath against a hard surface. **DO NOT use free hand to press the sheath over the needle.**



- 6 Dispose of Venipuncture Needle-Pro® device into an approved sharps container.

